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Ziegelei 1

D-72336 Balingen

E-Mail: [info@kern-sohn.com](mailto:info@kern-sohn.com)

Tel: +49-[0]7433- 9933-0

Fax: +49-[0]7433-9933-149

Internet: [www.kern-sohn.com](http://www.kern-sohn.com)

## **KERN MSF**

Type MSF 200N

Version 1.8

2018-03

D	2
GB	7
F	12
BG	17
CZ	22
DK	27
E	32
EST	37
FIN	42
GR	47
H	52
HR	57
I	62
LT	67
LV	72
NL	77
NO	82
P	87
PL	92
RO	97
SE	102
SK	107
SLO	112

**MSF-BA-EU-1818**

# KERN MSF

Version 1.8 2018-03

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# KERN MSF

Version 1.8 2018-03

## Operating instructions Mechanical height measuring rod

### 1 Technical Data

KERN (Type) MSF 200N			
Trademark	MSF 200	Storage temperature	- 20°C ... + 60°C
Readability	1 mm / 1/8 inch	Net weight	ca. 700 g
Measuring range	60 – 205 cm / 24-80 inch	Admitted as medical product as per Guideline 93/42/EWG ( CE 0297 )	Klasse I mit Messfunktion
Tolerance	10 mm	Combinable scales	<b>KERN MTS-M</b> <b>KERN MPS-PM</b> <b>KERN MPB-P</b>
Dimensions (D x H x W)	60 x 290 x 930 mm		
Temperature range	+10°C ... +40°C		

### 2 Declaration of conformity

To view the current EC/EU Declaration of Conformity go to:

[www.kern-sohn.com/ce](http://www.kern-sohn.com/ce)

### 3 Explanation of the graphic symbols

This marking shows that this weighing balance is in conformity with EU Directive 93/42/EEC and inside the European Community is classified as medical device.

SN WY 170563

Designation of the serial number of every device, applied at the device and on the packaging.  
(Number as an example)

2018-03

Designation of the serial number of every device, applied at the device and on the packaging.  
Number here as example

„Attention observe accompanying document“, or  
„Follow operating instructions“

Follow operating instructions

Kern & Sohn GmbH  
D-72336 Balingen, Germany  
www.kern-sohn.com

Identification of the manufacturer of the medical product including address

## 4 Basic Information (General)

This height measuring rod according to guideline 93/42/EWG is provided for determination the body height in medical practice and for medical survey, examination and treatment.

### Defined purpose

#### Indication:

- Determining the body height in the medical practice area.
- At an upright standing person, or at a lying baby, the height measuring rod is applied to head and feet in a manner that on the display or scale the body height value can be read.

#### Contraindication:

No contraindication known.

### Proper use

This height measuring rod is used to determine the height of persons while standing upright, or of lying babies, according to its model, in medical treatment localities. This height measuring rod is suitable to detect, prevent and survey diseases.

Ensure that the height measuring rod may only come into contact only with uninjured skin.

### Improper Use

The structure of the height measuring rod may not be modified. This may result in incorrect measuring results, safety-related defects as well as destruction. The height measuring rod may only be used according to the described conditions. Other areas of use must be released by KERN in writing.

### Warranty

Warranty claims shall be voided in case

Our conditions in the operation manual are ignored

The appliance is used outside the described uses

The appliance is modified or opened

Mechanical damage or damage by media, liquids, natural wear and tear. • Non correct implantation or assembly.

### Test device control

For height measuring rods, we recommend a metrological examination of the accuracy of the measuring rod, however, this is not mandatory as the determination of human body height involves rather large, intrinsic inaccuracies.

## 5 Basic Safety Precautions

⇒ Carefully read this operation manual before setup and commissioning, even if you are already familiar with KERN balances.

⇒ All language versions contain a non-binding translation. The original German is binding.

### Staff training

The medical staff must apply and follow the operating instructions for proper use and care of the product.

### Avoiding contamination

In order to avoid crossed contamination (fungal disease,...) the parts coming into contact with the patient must be cleared regularly.

Recommendation:

After every application which could have a potential contamination as consequence (e.g. direct contact with the skin).

### Safety Instructions

- Observe operating instructions.
- Please use no other than the supplied fixtures for mounting.
- During assembling, ensure that screws are tightened properly and that the height measuring rod has been assembled correctly.
- For medical application recheck correctness of scale after assembly with the help of a calibrated length measuring device. (see chap. 6 „Checking the scale“)
- To prevent risk of injury, ensure that the head stop is returned to folded-in position after each use.
- The body height measuring emits only reliable values, when the back and the head are straightly aligned, nevertheless the body height of a person may change about some cm during a day.

## 6 Unpacking/installing

### Testing upon acceptance

Please check packaging as well as the device on receipt for possibly visible external damages.

### Packaging/Return

Keep all parts of the original packaging for a possibly required return. Only use original packaging for returning.

### Checking the graduation

After the assembly of the height measuring rod, the graduation must be checked using an external measuring rod. For this put an external, verified measuring rod with the respective accurateness on the weighing plate or, in case of wall assembly, vertically on the floor. The height measuring rod must be shifted in a way that the horizontally unfolded flap touches the external measuring rod with the lower side at 150 cm.

Information:

The measure of the external measuring rod is not congruent with the graduation of the height measuring rod, it is displaced by some cm, as the offset between the flap and the reading mark thus will be considered.

If the measure does not match exactly, loosen again slightly and shift a little the screws used for assembly and then check again the measure of 150 cm.

### Scope of supply

2 x bracket

4 x dowel, "wall bracket"

4 x screw, "scales stand bracket"

2 x screw „Measuring rod on tripod of scales“


4 x screws „Measuring rod on wall with bracket“


Operating instructions

## 7 Installation

**Wall assembly:** Set marking at 54 cm above the floor, using a verified length measuring device.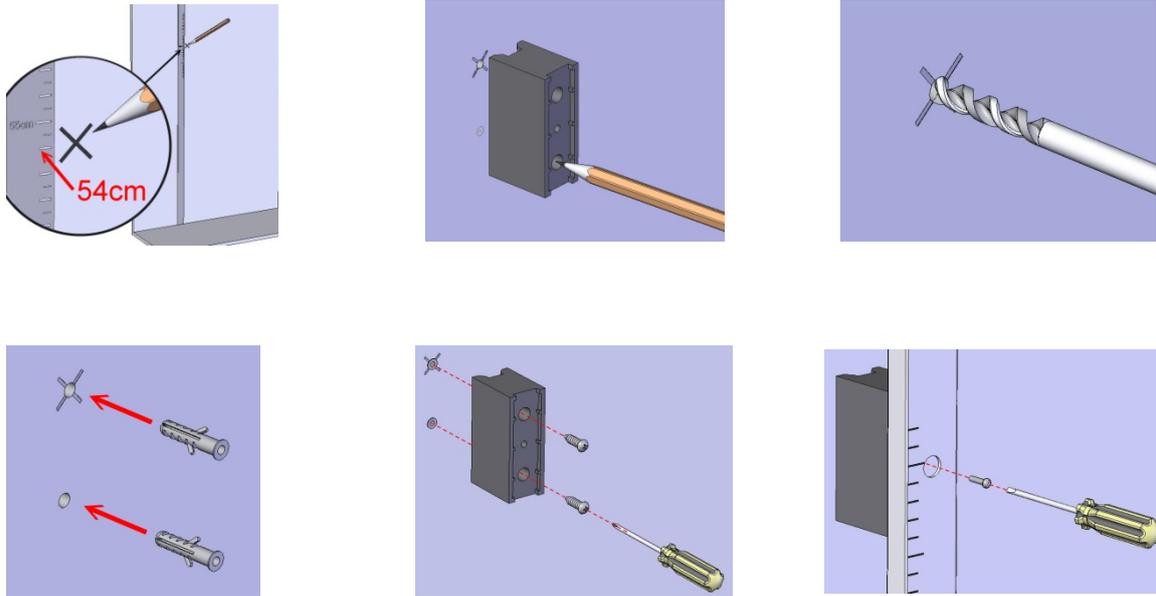

**How to attach to KERN scale:**

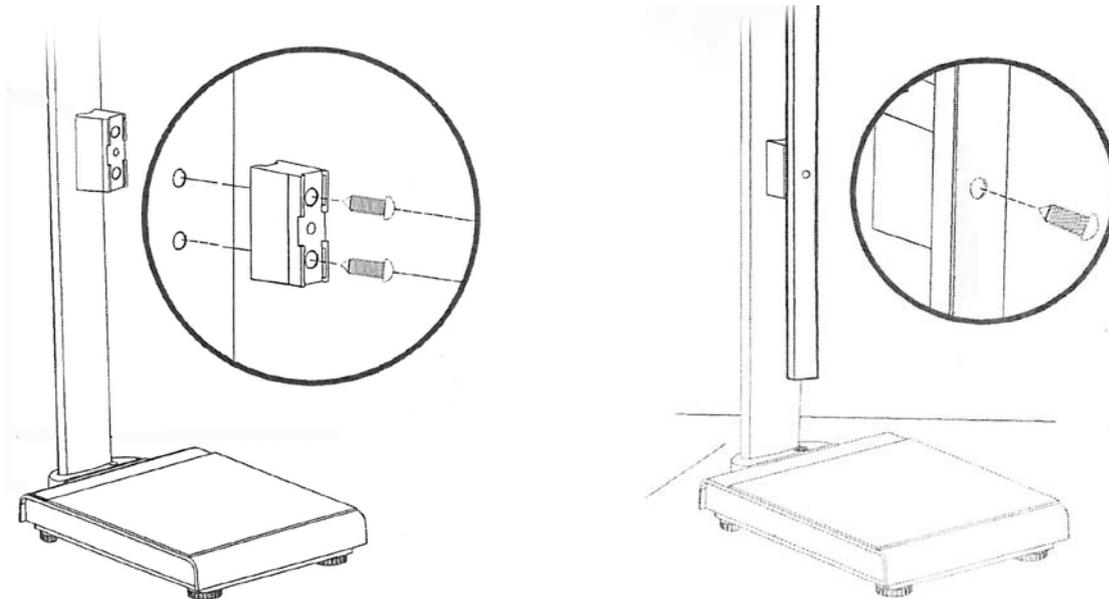Use the 2 screws for screwing the bracket into the supplied insert nuts on the stand of the scales.


Extend the height measuring rod and tighten it in the lower hole of the bracket with the help of the ap-pertaining screw.

**The body height scale can be mounted in the same way at the back of the support stand.**


## 8 Body height measurement

### Body height above 120 cm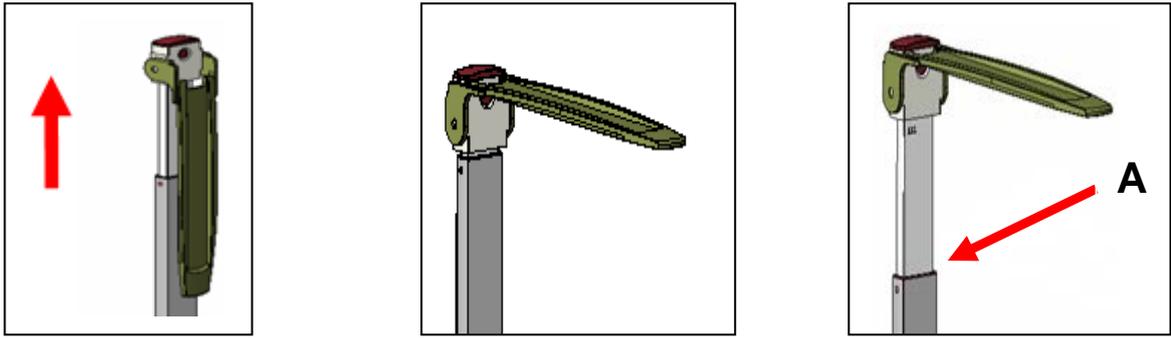

1. Pull measuring rod upwards
2. Move head stop up into horizontal position until it locks.
3. Carefully push the measuring rod down until the head stop gently touches the head of the aligned person. Read body height on the measuring rod (marking A).

### Body height until 120 cm

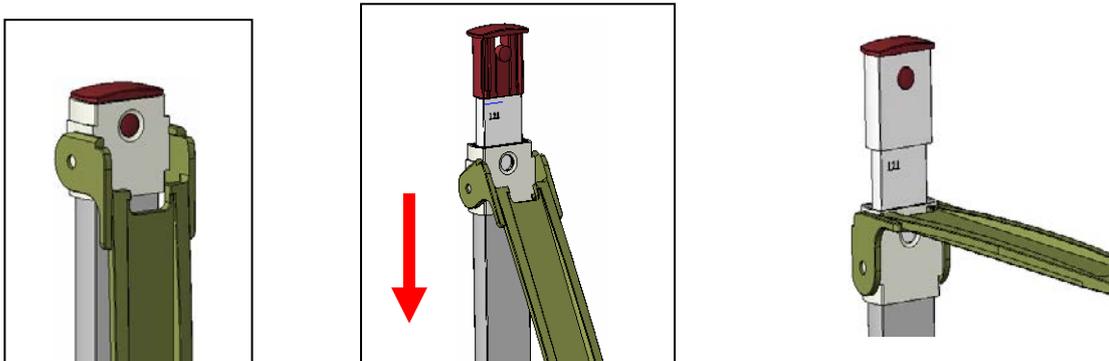4. Press locking and keep it pressed.
5. Shift head stop downwards.  
Then move head stop up into horizontal position until it locks.
6. Carefully push the measuring rod down until the head stop gently touches the head of the aligned person. Read body height on the measuring rod (marking B).

## 9 Cleaning, Maintenance, Disposal

<b>Cleaning</b>	The body height scale should be cleaned with household detergents or commercially available disinfectants, e.g. 70% isopropanol. We recommend a disinfectant suitable for wiping disinfection. Please follow manufacturer's instructions.
<b>Disinfection</b>	Do not use any polishing or aggressive cleaning agents (white spirit, petrol etc.) as they may damage the high-quality surface.
<b>Sterilisation</b>	Sterilisation of the appliance not allowed.
<b>Service, maintenance</b>	The appliance may only be opened by trained service technicians who are authorized by KERN.
<b>Disposal</b>	Disposal of packaging and appliance must be carried out by operator according to valid national or regional law of the location where the appliance is used..

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